



Outdoor Defibrillator Cabinet with Alarm

Alarm Instructions:

The alarm is shipped with a 9V battery installed and an alarm key in place. When the alarm key is in place, the alarm is **off**. With the key in place, the alarm will not sound when the cabinet door is opened or closed. You may leave the key in place if you do not see a need for an alarm in your office.

To Activate the Alarm:

1. Open the cabinet door
2. Remove the alarm key from the bottom of the red alarm located inside of the cabinet
3. Place the AED into the cabinet and close the door. *Be sure to close the cabinet door in one complete movement. The door is held closed by a contact magnet along the door's edge.*

The alarm is now set. The next time the door is opened the alarm will sound. The alarm will continue to sound for 3 minutes or until the key is replaced into the keyhole in the alarm box.

To Install The Mounting Brackets:

A mounting bracket and screw pack are provided with the DSG-500 Outdoor Cabinet. To assemble, turn the enclosure upside down on a flat surface, place the mounting bracket over the square boss (horizontally or vertically) and fasten the mounting bracket with the screws provided.